

## **Instructions for EVAsoft Integration**

MouthWatch cameras easily integrate with EVAsoft Imaging software. Use the MouthWatch camera to capture images of the patient and store them directly into the patient record.

### **Step 1 – Download & Install the Button Capture Software**

- Go to: [www.mymouthwatch.com/support](http://www.mymouthwatch.com/support) and download our Button Capture Software.
- View the installation guide
- Install the Button Capture Software program, following all steps in the guide
- Right click on the MouthWatch Capture icon in the system tray
- Select “TWAIN” from the Button Capture Software menu

### **Step 2 – Plug in the MouthWatch Camera**

Make sure that you plug the USB connector into a powered USB port. Sometimes, these are located on the front or back of the computer. If your monitor has USB ports, these may not be powered.

### **Step 3 – Windows Recognizes the Camera**

Once the MouthWatch camera is plugged in, you should see a small alert on the bottom saying that New Hardware “USB Video Device” was recognized.

### **Step 4 – Install the TWAIN Driver**

You will need to copy the TWAIN driver from the MouthWatch Capture program folder into the C:\Windows\twain\_32 folder.


1. Open C:\Program Files (x86)\MouthWatch Capture\Prerequisites
2. Copy the file VideoDS.ds
3. Open C:\Windows\twain\_32
4. Paste the VideoDS.ds file into this folder

### **Step 5 – Launch EVAsoft**

- Select a Patient from the list that displays upon launch


## Step 6 – Select Imaging Device

- Under Image Devices, select TWAIN – Intraoral Camera Twain

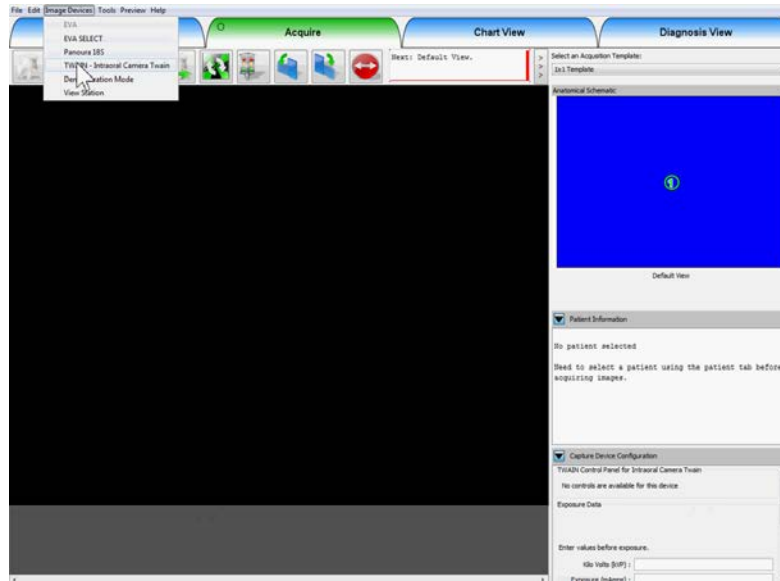


## Step 7 – Start Exposure

- Click on the tooth icon to begin the capture

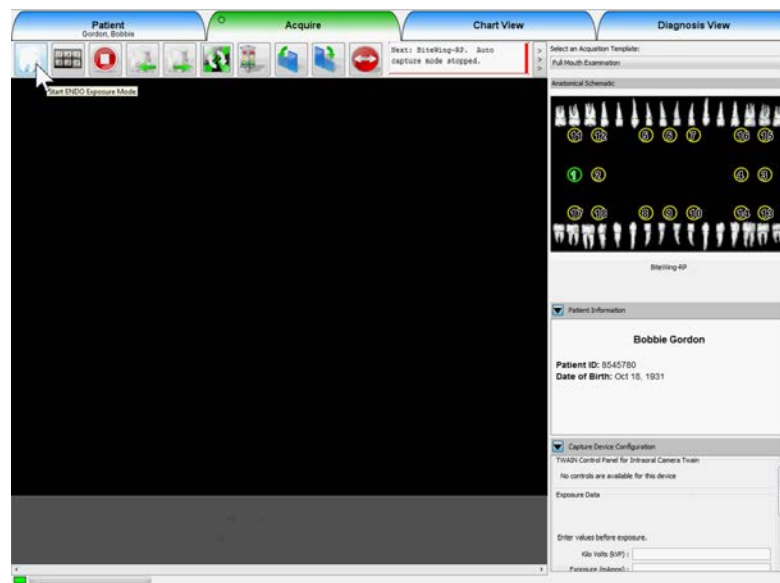

- This will launch the TWAIN capture window

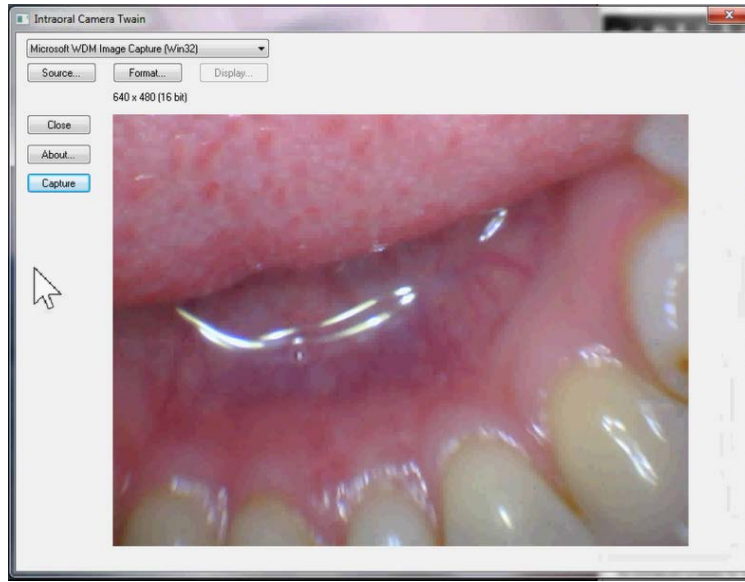

- Pressing the capture button on the camera will take a photo and store it to the patient chart

## Step 7 – All Set!